

Item Catalogue

NUPCO CODE: 4232150012800 | SN: 601

NUPCO ITEM DESCRIPTION

PLATE, VARIABLE ANGLE ANATOMIC PROXIMAL OLECRANON LEFT OR RIGHT ASSORTED TYPES AND SIZES AS REQUESTED BY END USER

VENDOR PRODUCT DESCRIPTION

Elbow Plating System olecranon plates

SKU CODES INCLUDED FOR THIS NUPCO CODE

- 70-0302
- 70-0303
- 70-0304
- 70-0305
- 70-0306
- 70-0307
- 70-0308
- 70-0309
- 70-0310
- 70-0311
- 70-0312
- 70-0313
- 70-0314
- 70-0315
- 70-0316
- 70-0317



Surgical Technique



Olecranon Plate Surgical Technique [continued]



Figure 3

3 Nonlocking Distal Screw Placement

With provisional reduction confirmed, drill with the 2.8 mm Quick Release Drill (80-0387) through a slotted hole, distal to the fracture site and into the ulnar shaft. Use the Depth Gauge 6–65 mm (80-0623) to measure for screw length. Connect the T15 Stick Fit Hexalobe Driver (80-0760) to the Medium Ratcheting Driver Handle (80-0663) and insert the appropriate 3.5 mm Nonlocking Hexalobe Screw (30-02XX) or 3.5 mm Nonlocking Hexalobe CoCr Screw (3063-350XX). Tighten the screw partially to allow for later compression. A 3.5 mm Cortical Screw Bone Tap (MS-LTT35) is available for patients with dense bone.

Note: When implanting the Narrow 5-Hole Olecranon Plates (70-0316 or 70-0317), only the 2.7 mm locking and nonlocking hexalobe screws and associated instrumentation may be used throughout all plate holes.

Note: 3.0 mm or 3.5 mm Nonlocking Hexalobe Screws (30-03XX or 30-02XX) or 3.5 mm Nonlocking Hexalobe CoCr Screws (3063-350XX) can be used in the shaft of the plate.

Screw Diameter	Drill Diameter
2.7 mm	2.0 mm
3.0 mm	2.3 mm
3.5 mm	2.8 mm




Figure 4

4 Proximal Locking Screw Placement

To insert two 2.7 mm Locking Hexalobe Screws (30-03XX) or 2.7 mm Locking Hexalobe CoCr Screws (3060-270XX) into the proximal holes on either side of the 2.0 mm wire, begin by threading the 2.0 mm Locking Drill Guide 4–32 mm (80-0621) through the proximal targeting guide and into the most proximal locking holes. While drilling with the 2.0 mm Quick Release Drill (80-0318), be sure not to exit the bone. Drill depth may be read directly off the laser line on the drill through the locking drill guide or with the 2.0 mm Depth Probe (80-0643). To insert the appropriate 2.7 mm locking hexalobe screw, connect the T8 Stick Fit Hexalobe Driver (80-0759) to the Medium Ratcheting Driver Handle (80-0663).

Note: When using the T8 driver, care should be taken not to overtighten the screw or apply more torque than necessary to seat the locking screw into the plate. Screws should be tightened by hand and not under power.

Ordering Information [continued]

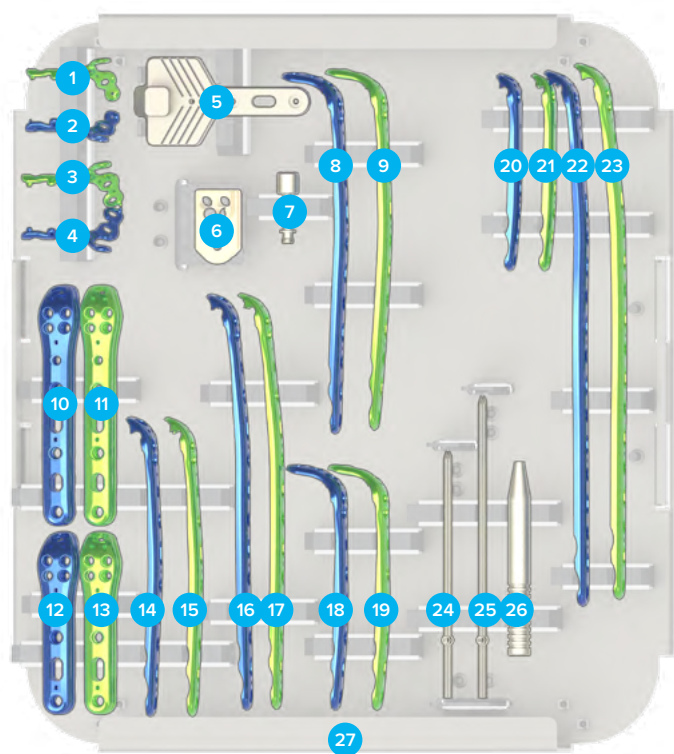

Tray Components

Coronoid Plates

3	Coronoid Plate, Standard, Right	70-0414
4	Coronoid Plate, Standard, Left	70-0413

Instrumentation

5	Olecranon Plate Osteotomy Cutting Jig	80-0653
6	Olecranon Plate Prox Targeting Guide	80-0654
7	Locking Bolt: M4	80-0652

Olecranon Plates

8	Olecranon Plate, Extended, 9-hole, Left (130 mm)	70-0314
9	Olecranon Plate, Extended, 9-hole, Right (130 mm)	70-0315
10	Olecranon Plate, Standard, 5-hole, Left (90 mm)	70-0304
11	Olecranon Plate, Standard, 5-hole, Right (90 mm)	70-0305
12	Olecranon Plate, Standard, 3-hole, Left (65 mm)	70-0302

Tray Components

13	Olecranon Plate, Standard, 3-hole, Right (65 mm)	70-0303
14	Olecranon Plate, Standard, 7-hole, Left (110 mm)	70-0306
15	Olecranon Plate, Standard, 7-hole, Right (110 mm)	70-0307
16	Olecranon Plate, Standard, 11-hole, Left (150 mm)	70-0308
17	Olecranon Plate, Standard, 11-hole, Right (150 mm)	70-0309
18	Olecranon Plate, Extended, 5-hole, Left (90 mm)	70-0312
19	Olecranon Plate, Extended, 5-hole, Right (90 mm)	70-0313

Instrumentation

24	70 mm Tension Band Pin	30-0098
25	90 mm Tension Band Pin	30-0099
26	Tension Band Pin Snapper	80-0411
27	Elbow System Olecranon-Coronoid Platter	80-0677

Optional Components from Elbow Plating System

Coronoid Plates

1	Coronoid Plate, Small, Right	70-0416
2	Coronoid Plate, Small, Left	70-0415

Olecranon Plates

20	Olecranon Plate, Narrow, 5-hole, Left (85 mm)	70-0316
21	Olecranon Plate, Narrow, 5-hole, Right (85 mm)	70-0317
22	Olecranon Plate, Standard, 15-hole, Left (190 mm)	70-0310
23	Olecranon Plate, Standard, 15-hole, Right (190 mm)	70-0311