

Item Catalogue

NUPCO CODE: 4232150020200 | SN: 689

NUPCO ITEM DESCRIPTION

SCREW, LOCKED HEAD 3.5MM OR EQUIVALENT FOR TITANIUM HOOK CLAVICLE PLATE, ASSORTED SIZES

VENDOR PRODUCT DESCRIPTION

Acumed Clavicle Hook Plate 3.5 mm locking screw families

SKU CODES INCLUDED FOR THIS NUPCO CODE

- 30-0232
- 30-0233
- 30-0234
- 30-0235
- 30-0236
- 30-0237
- 30-0238
- 30-0239
- 30-0240
- 30-0241
- 30-0242
- 30-0243
- 30-0244
- 30-0245
- 30-0246
- 30-0247
- 30-0248
- 30-3642
- 30-3644
- 30-3646
- 30-3648
- 30-0250
- 30-0251
- 30-0252
- 30-0253

 acumed[®]
Clavicle Hook Plating System

Surgical Technique



Ordering Information [continued]

Screws

3.5 mm Locking Hexalobe Screws

3.5 mm x 8 mm Locking Hexalobe Screw	30-0232
3.5 mm x 10 mm Locking Hexalobe Screw	30-0233
3.5 mm x 12 mm Locking Hexalobe Screw	30-0234
3.5 mm x 14 mm Locking Hexalobe Screw	30-0235
3.5 mm x 16 mm Locking Hexalobe Screw	30-0236
3.5 mm x 18 mm Locking Hexalobe Screw	30-0237
3.5 mm x 20 mm Locking Hexalobe Screw	30-0238
3.5 mm x 22 mm Locking Hexalobe Screw	30-0239
3.5 mm x 24 mm Locking Hexalobe Screw	30-0240
3.5 mm x 26 mm Locking Hexalobe Screw	30-0241

3.5 mm Nonlocking Hexalobe Screws

3.5 mm x 8 mm Nonlocking Hexalobe Screw	30-0255
3.5 mm x 10 mm Nonlocking Hexalobe Screw	30-0256
3.5 mm x 12 mm Nonlocking Hexalobe Screw	30-0257
3.5 mm x 14 mm Nonlocking Hexalobe Screw	30-0258
3.5 mm x 16 mm Nonlocking Hexalobe Screw	30-0259
3.5 mm x 18 mm Nonlocking Hexalobe Screw	30-0260
3.5 mm x 20 mm Nonlocking Hexalobe Screw	30-0261
3.5 mm x 22 mm Nonlocking Hexalobe Screw	30-0262
3.5 mm x 24 mm Nonlocking Hexalobe Screw	30-0263
3.5 mm x 26 mm Nonlocking Hexalobe Screw	30-0264

3.5 mm (Nonlocking) Cortical (Hex) Screws

3.5 mm x 8 mm Cortical Screw	CO-3080
3.5 mm x 10 mm Cortical Screw	CO-3100
3.5 mm x 12 mm Cortical Screw	CO-3120
3.5 mm x 14 mm Cortical Screw	CO-3140
3.5 mm x 16 mm Cortical Screw	CO-3160
3.5 mm x 18 mm Cortical Screw	CO-3180
3.5 mm x 20 mm Cortical Screw	CO-3200
3.5 mm x 22 mm Cortical Screw	CO-3220
3.5 mm x 24 mm Cortical Screw	CO-3240
3.5 mm x 26 mm Cortical Screw	CO-3260

3.5 mm Locking Cortical (Hex) Screws

3.5 mm x 8 mm Locking Cortical Screw	COL-3080
3.5 mm x 10 mm Locking Cortical Screw	COL-3100
3.5 mm x 12 mm Locking Cortical Screw	COL-3120
3.5 mm x 14 mm Locking Cortical Screw	COL-3140
3.5 mm x 16 mm Locking Cortical Screw	COL-3160
3.5 mm x 18 mm Locking Cortical Screw	COL-3180
3.5 mm x 20 mm Locking Cortical Screw	COL-3200
3.5 mm x 22 mm Locking Cortical Screw	COL-3220
3.5 mm x 24 mm Locking Cortical Screw	COL-3240
3.5 mm x 26 mm Locking Cortical Screw	COL-3260

Ordering Information [continued]

Sterile Screws

Sterile 3.5 mm Locking Hexalobe Screws

3.5 mm x 8 mm Locking Hexalobe Screw	30-0232-S
3.5 mm x 10 mm Locking Hexalobe Screw	30-0233-S
3.5 mm x 12 mm Locking Hexalobe Screw	30-0234-S
3.5 mm x 14 mm Locking Hexalobe Screw	30-0235-S
3.5 mm x 16 mm Locking Hexalobe Screw	30-0236-S
3.5 mm x 18 mm Locking Hexalobe Screw	30-0237-S
3.5 mm x 20 mm Locking Hexalobe Screw	30-0238-S
3.5 mm x 22 mm Locking Hexalobe Screw	30-0239-S
3.5 mm x 24 mm Locking Hexalobe Screw	30-0240-S
3.5 mm x 26 mm Locking Hexalobe Screw	30-0241-S

Sterile 3.5 mm Nonlocking Hexalobe Screws

3.5 mm x 8 mm Nonlocking Hexalobe Screw	30-0255-S
3.5 mm x 10 mm Nonlocking Hexalobe Screw	30-0256-S
3.5 mm x 12 mm Nonlocking Hexalobe Screw	30-0257-S
3.5 mm x 14 mm Nonlocking Hexalobe Screw	30-0258-S
3.5 mm x 16 mm Nonlocking Hexalobe Screw	30-0259-S
3.5 mm x 18 mm Nonlocking Hexalobe Screw	30-0260-S
3.5 mm x 20 mm Nonlocking Hexalobe Screw	30-0261-S
3.5 mm x 22 mm Nonlocking Hexalobe Screw	30-0262-S
3.5 mm x 24 mm Nonlocking Hexalobe Screw	30-0263-S
3.5 mm x 26 mm Nonlocking Hexalobe Screw	30-0264-S

Sterile 3.5 mm (Nonlocking) Cortical (Hex) Screws


3.5 mm x 8 mm Cortical Screw	CO-3080-S
3.5 mm x 10 mm Cortical Screw	CO-3100-S
3.5 mm x 12 mm Cortical Screw	CO-3120-S
3.5 mm x 14 mm Cortical Screw	CO-3140-S
3.5 mm x 16 mm Cortical Screw	CO-3160-S
3.5 mm x 18 mm Cortical Screw	CO-3180-S
3.5 mm x 20 mm Cortical Screw	CO-3200-S
3.5 mm x 22 mm Cortical Screw	CO-3220-S
3.5 mm x 24 mm Cortical Screw	CO-3240-S
3.5 mm x 26 mm Cortical Screw	CO-3260-S

Sterile 3.5 mm Locking Cortical (Hex) Screws

3.5 mm x 8 mm Locking Cortical Screw	COL-3080-S
3.5 mm x 10 mm Locking Cortical Screw	COL-3100-S
3.5 mm x 12 mm Locking Cortical Screw	COL-3120-S
3.5 mm x 14 mm Locking Cortical Screw	COL-3140-S
3.5 mm x 16 mm Locking Cortical Screw	COL-3160-S
3.5 mm x 18 mm Locking Cortical Screw	COL-3180-S
3.5 mm x 20 mm Locking Cortical Screw	COL-3200-S
3.5 mm x 22 mm Locking Cortical Screw	COL-3220-S
3.5 mm x 24 mm Locking Cortical Screw	COL-3240-S
3.5 mm x 26 mm Locking Cortical Screw	COL-3260-S

Note: To learn more about the full line of Acumed innovative surgical solutions, please contact your authorized Acumed distributor, call 888.627.9957, or visit www.acumed.net.

Surgical Technique

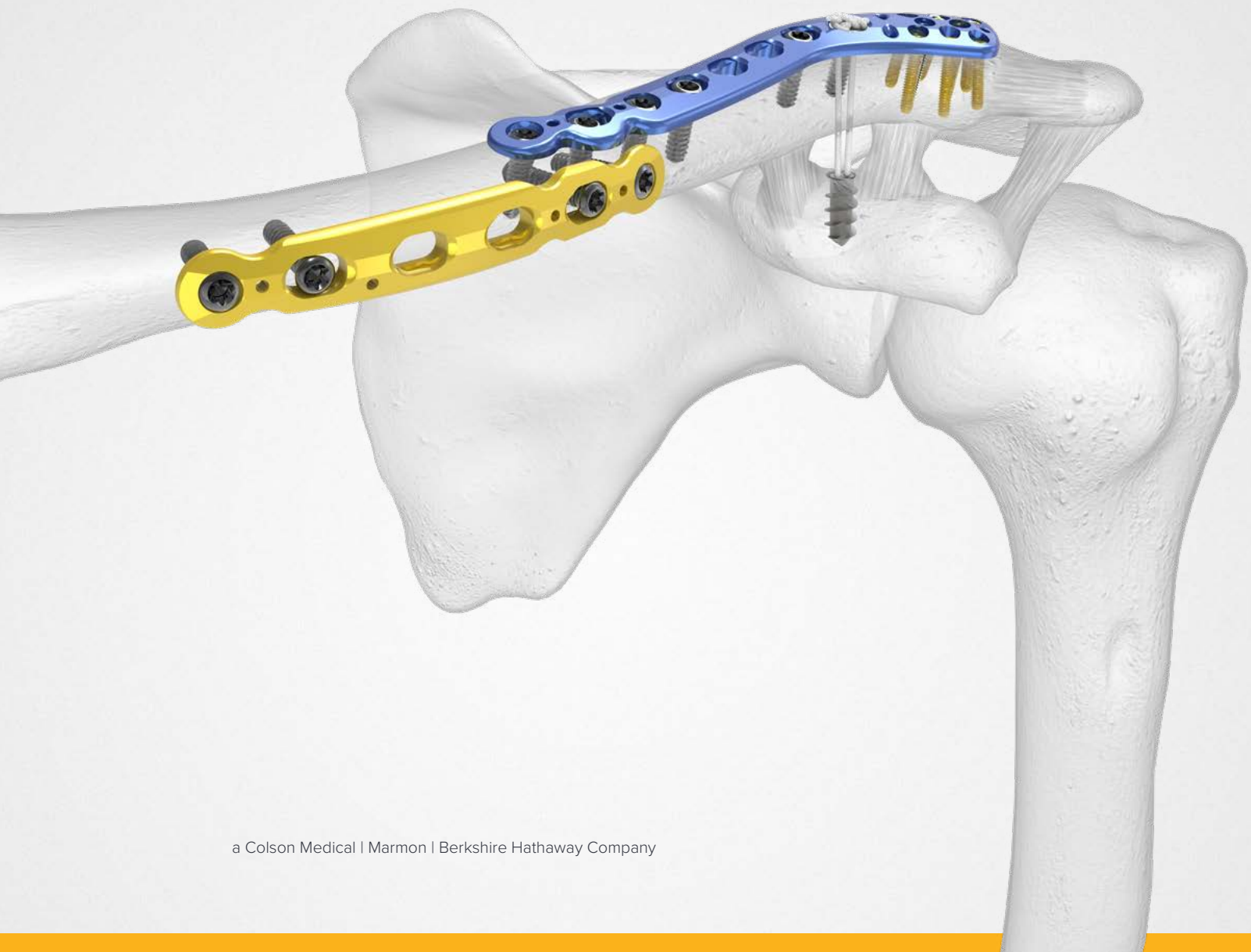

Superior Distal Clavicle Plate Surgical Technique [continued]

4 Plate Placement

Once the plate's ideal positioning has been selected, it is provisionally stabilized to the clavicle with .045" x 6" ST Guide Wire (WS-1106ST) or .059" x 5" ST Guide Wire (WS-1505ST) K-wires. Under radiographic evaluation, the most lateral K-wire hole of each Superior Distal Clavicle Plate (70-0XXX) affords the opportunity to verify that the placement of the screws will not protrude into the AC joint by inserting a K-wire to confirm plate placement.

Note: The Reduction Forceps With Serrated Jaw (PL-CL04) should be used for plate placement and are not designed to be used to reduce the plate to the bone or to hold the plate while attempting to bend or contour it to match the patient's anatomy. Plate Benders (PL-204X) are available in the event that plate contouring is required to achieve the desired fit to the clavicle.


Caution: If bending of the plate is necessary, please observe the following:

- ▶ Do not bend plates more than 30 degrees
- ▶ Bend radii should be greater than 1 inch
- ▶ Do not bend more than once
- ▶ Avoid bending across locking holes

Figure 5



Superior Distal Clavicle Plate Surgical Technique [continued]

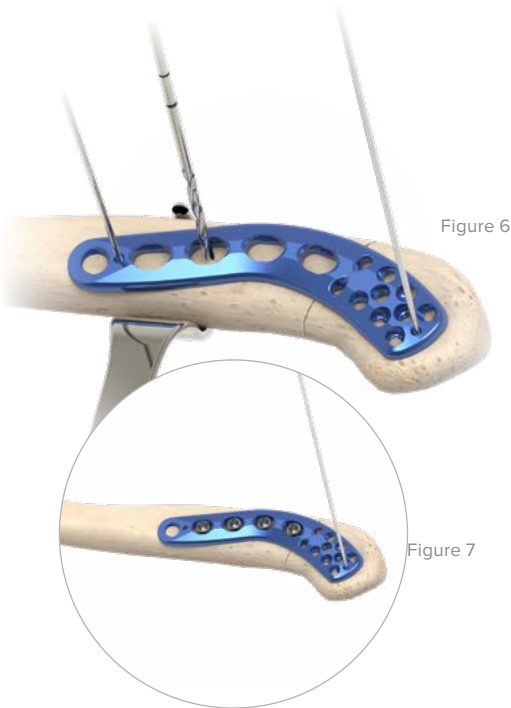

5 Nonlocking Screw Insertion

For early stability, place the first two screws medial and lateral to the fracture site. If bicortical screws are used, take precautions to avoid over-penetration of the inferior cortex. Place the Clavicle Retractor (PL-CL03) under the inferior surface of the clavicle to protect the neurovascular structures from overpenetration when drilling.

Caution: Replace the drill if it comes in contact with the clavicle retractor.

If drilling for 3.5 mm size screws, the Offset Drill Guide (PL-2095) is available to assist in providing compression. The green side of the drill guide is neutral, while the gold side will generate compression when used in a compression slot on the plate.

Screw Size Option	Drill	Depth Gauge	Driver	Driver Handle
3.5 mm Nonlocking Hexalobe Screw (30-02XX)	2.8 mm Quick Release Drill (80-0387)	Depth Gauge 6–65 mm (80-0623)	T15 Stick Fit Hexalobe Driver (80-0760)	Large Cannulated Quick Release Driver Handle (MS-3200)
3.0 mm Nonlocking Hexalobe Screw (30-03XX)	2.3 mm Quick Release Drill (80-0627)	Depth Gauge 6–65 mm (80-0623)	T15 Stick Fit Hexalobe Driver (80-0760)	Large Cannulated Quick Release Driver Handle (MS-3200)
Optional 3.5 mm (Nonlocking) Cortical (Hex) Screw (CO-3XX0)	2.8 mm x 5" Quick Release Drill (MS-DC28)	6–70 mm Depth Gauge, 2 mm Increments (MS-9022)	2.5 mm Quick Release Hex Driver (HPC-0025)	Large Cannulated Quick Release Driver Handle (MS-3200)
Optional 2.7 mm (Nonlocking) Cortical (Hex) Screw (CO-27XX)	2.0 mm x 5" Quick Release Drill (MS-DC5020)	6–70 mm Depth Gauge, 2 mm Increments (MS-9022)	2.5 mm Quick Release Hex Driver (HPC-0025)	Large Cannulated Quick Release Driver Handle (MS-3200)

Ordering Information [continued]

Screws

3.5 mm Locking Hexalobe Screws

3.5 mm x 8 mm Locking Hexalobe Screw	30-0232
3.5 mm x 10 mm Locking Hexalobe Screw	30-0233
3.5 mm x 12 mm Locking Hexalobe Screw	30-0234
3.5 mm x 14 mm Locking Hexalobe Screw	30-0235
3.5 mm x 16 mm Locking Hexalobe Screw	30-0236
3.5 mm x 18 mm Locking Hexalobe Screw	30-0237
3.5 mm x 20 mm Locking Hexalobe Screw	30-0238
3.5 mm x 22 mm Locking Hexalobe Screw	30-0239
3.5 mm x 24 mm Locking Hexalobe Screw	30-0240
3.5 mm x 26 mm Locking Hexalobe Screw	30-0241

3.5 mm Nonlocking Hexalobe Screws

3.5 mm x 8 mm Nonlocking Hexalobe Screw	30-0255
3.5 mm x 10 mm Nonlocking Hexalobe Screw	30-0256
3.5 mm x 12 mm Nonlocking Hexalobe Screw	30-0257
3.5 mm x 14 mm Nonlocking Hexalobe Screw	30-0258
3.5 mm x 16 mm Nonlocking Hexalobe Screw	30-0259
3.5 mm x 18 mm Nonlocking Hexalobe Screw	30-0260
3.5 mm x 20 mm Nonlocking Hexalobe Screw	30-0261
3.5 mm x 22 mm Nonlocking Hexalobe Screw	30-0262
3.5 mm x 24 mm Nonlocking Hexalobe Screw	30-0263
3.5 mm x 26 mm Nonlocking Hexalobe Screw	30-0264

Hexalobe Short Screw Caddies

3.5 mm Hexalobe Short Screw Caddy	80-0843
3.5 mm Hexalobe Short Screw Caddy Lid	80-0856
3.0 mm Hexalobe Short Screw Caddy	80-1066
3.0 mm Hexalobe Short Screw Caddy Lid	80-1067

3.0 mm Locking Hexalobe Screws

3.0 mm x 8 mm Locking Hexalobe Screw	30-0278
3.0 mm x 10 mm Locking Hexalobe Screw	30-0279
3.0 mm x 12 mm Locking Hexalobe Screw	30-0280
3.0 mm x 14 mm Locking Hexalobe Screw	30-0281
3.0 mm x 16 mm Locking Hexalobe Screw	30-0282
3.0 mm x 18 mm Locking Hexalobe Screw	30-0283
3.0 mm x 20 mm Locking Hexalobe Screw	30-0284
3.0 mm x 22 mm Locking Hexalobe Screw	30-0285
3.0 mm x 24 mm Locking Hexalobe Screw	30-0286
3.0 mm x 26 mm Locking Hexalobe Screw	30-0287

3.0 mm Nonlocking Hexalobe Screws

3.0 mm x 8 mm Nonlocking Hexalobe Screw	30-0301
3.0 mm x 10 mm Nonlocking Hexalobe Screw	30-0302
3.0 mm x 12 mm Nonlocking Hexalobe Screw	30-0303
3.0 mm x 14 mm Nonlocking Hexalobe Screw	30-0304
3.0 mm x 16 mm Nonlocking Hexalobe Screw	30-0305
3.0 mm x 18 mm Nonlocking Hexalobe Screw	30-0306
3.0 mm x 20 mm Nonlocking Hexalobe Screw	30-0307
3.0 mm x 22 mm Nonlocking Hexalobe Screw	30-0308
3.0 mm x 24 mm Nonlocking Hexalobe Screw	30-0309
3.0 mm x 26 mm Nonlocking Hexalobe Screw	30-0310

Ordering Information [continued]

Sterile Screws

3.5 mm Locking Hexalobe Screws

3.5 mm x 8 mm Locking Hexalobe Screw	30-0232-S
3.5 mm x 10 mm Locking Hexalobe Screw	30-0233-S
3.5 mm x 12 mm Locking Hexalobe Screw	30-0234-S
3.5 mm x 14 mm Locking Hexalobe Screw	30-0235-S
3.5 mm x 16 mm Locking Hexalobe Screw	30-0236-S
3.5 mm x 18 mm Locking Hexalobe Screw	30-0237-S
3.5 mm x 20 mm Locking Hexalobe Screw	30-0238-S
3.5 mm x 22 mm Locking Hexalobe Screw	30-0239-S
3.5 mm x 24 mm Locking Hexalobe Screw	30-0240-S
3.5 mm x 26 mm Locking Hexalobe Screw	30-0241-S

3.5 mm Nonlocking Hexalobe Screws

3.5 mm x 8 mm Nonlocking Hexalobe Screw	30-0255-S
3.5 mm x 10 mm Nonlocking Hexalobe Screw	30-0256-S
3.5 mm x 12 mm Nonlocking Hexalobe Screw	30-0257-S
3.5 mm x 14 mm Nonlocking Hexalobe Screw	30-0258-S
3.5 mm x 16 mm Nonlocking Hexalobe Screw	30-0259-S
3.5 mm x 18 mm Nonlocking Hexalobe Screw	30-0260-S
3.5 mm x 20 mm Nonlocking Hexalobe Screw	30-0261-S
3.5 mm x 22 mm Nonlocking Hexalobe Screw	30-0262-S
3.5 mm x 24 mm Nonlocking Hexalobe Screw	30-0263-S
3.5 mm x 26 mm Nonlocking Hexalobe Screw	30-0264-S

3.0 mm Locking Hexalobe Screws

3.0 mm x 8 mm Locking Hexalobe Screw	30-0278-S
3.0 mm x 10 mm Locking Hexalobe Screw	30-0279-S
3.0 mm x 12 mm Locking Hexalobe Screw	30-0280-S
3.0 mm x 14 mm Locking Hexalobe Screw	30-0281-S
3.0 mm x 16 mm Locking Hexalobe Screw	30-0282-S
3.0 mm x 18 mm Locking Hexalobe Screw	30-0283-S
3.0 mm x 20 mm Locking Hexalobe Screw	30-0284-S
3.0 mm x 22 mm Locking Hexalobe Screw	30-0285-S
3.0 mm x 24 mm Locking Hexalobe Screw	30-0286-S
3.0 mm x 26 mm Locking Hexalobe Screw	30-0287-S

3.0 mm Nonlocking Hexalobe Screws

3.0 mm x 8 mm Nonlocking Hexalobe Screw	30-0301-S
3.0 mm x 10 mm Nonlocking Hexalobe Screw	30-0302-S
3.0 mm x 12 mm Nonlocking Hexalobe Screw	30-0303-S
3.0 mm x 14 mm Nonlocking Hexalobe Screw	30-0304-S
3.0 mm x 16 mm Nonlocking Hexalobe Screw	30-0305-S
3.0 mm x 18 mm Nonlocking Hexalobe Screw	30-0306-S
3.0 mm x 20 mm Nonlocking Hexalobe Screw	30-0307-S
3.0 mm x 22 mm Nonlocking Hexalobe Screw	30-0308-S
3.0 mm x 24 mm Nonlocking Hexalobe Screw	30-0309-S
3.0 mm x 26 mm Nonlocking Hexalobe Screw	30-0310-S

Ordering Information [continued]

Screws

4 3.5 mm Nonlocking Hexalobe Screws*

3.5 mm x 8 mm Nonlocking Hexalobe Screw	30-0255
3.5 mm x 10 mm Nonlocking Hexalobe Screw	30-0256
3.5 mm x 12 mm Nonlocking Hexalobe Screw	30-0257
3.5 mm x 14 mm Nonlocking Hexalobe Screw	30-0258
3.5 mm x 16 mm Nonlocking Hexalobe Screw	30-0259
3.5 mm x 18 mm Nonlocking Hexalobe Screw	30-0260
3.5 mm x 20 mm Nonlocking Hexalobe Screw	30-0261
3.5 mm x 22 mm Nonlocking Hexalobe Screw	30-0262
3.5 mm x 24 mm Nonlocking Hexalobe Screw	30-0263
3.5 mm x 26 mm Nonlocking Hexalobe Screw	30-0264
3.5 mm x 28 mm Nonlocking Hexalobe Screw	30-0265
3.5 mm x 30 mm Nonlocking Hexalobe Screw	30-0266
3.5 mm x 32 mm Nonlocking Hexalobe Screw	30-0267
3.5 mm x 34 mm Nonlocking Hexalobe Screw	30-0268
3.5 mm x 36 mm Nonlocking Hexalobe Screw	30-0269
3.5 mm x 38 mm Nonlocking Hexalobe Screw	30-0270
3.5 mm x 40 mm Nonlocking Hexalobe Screw	30-0271
3.5 mm x 42 mm Nonlocking Hexalobe Screw	30-3542
3.5 mm x 44 mm Nonlocking Hexalobe Screw	30-3544
3.5 mm x 46 mm Nonlocking Hexalobe Screw	30-3546
3.5 mm x 48 mm Nonlocking Hexalobe Screw	30-3548
3.5 mm x 50 mm Nonlocking Hexalobe Screw	30-0273
3.5 mm x 55 mm Nonlocking Hexalobe Screw	30-0274
3.5 mm x 60 mm Nonlocking Hexalobe Screw	30-0275
3.5 mm x 65 mm Nonlocking Hexalobe Screw	30-0276

5 3.5 mm Locking Hexalobe Screws*

3.5 mm x 8 mm Locking Hexalobe Screw	30-0232
3.5 mm x 10 mm Locking Hexalobe Screw	30-0233
3.5 mm x 12 mm Locking Hexalobe Screw	30-0234
3.5 mm x 14 mm Locking Hexalobe Screw	30-0235
3.5 mm x 16 mm Locking Hexalobe Screw	30-0236
3.5 mm x 18 mm Locking Hexalobe Screw	30-0237
3.5 mm x 20 mm Locking Hexalobe Screw	30-0238
3.5 mm x 22 mm Locking Hexalobe Screw	30-0239
3.5 mm x 24 mm Locking Hexalobe Screw	30-0240
3.5 mm x 26 mm Locking Hexalobe Screw	30-0241
3.5 mm x 28 mm Locking Hexalobe Screw	30-0242
3.5 mm x 30 mm Locking Hexalobe Screw	30-0243
3.5 mm x 32 mm Locking Hexalobe Screw	30-0244
3.5 mm x 34 mm Locking Hexalobe Screw	30-0245
3.5 mm x 36 mm Locking Hexalobe Screw	30-0246
3.5 mm x 38 mm Locking Hexalobe Screw	30-0247
3.5 mm x 40 mm Locking Hexalobe Screw	30-0248
3.5 mm x 42 mm Locking Hexalobe Screw	30-3642
3.5 mm x 44 mm Locking Hexalobe Screw	30-3644
3.5 mm x 46 mm Locking Hexalobe Screw	30-3646
3.5 mm x 48 mm Locking Hexalobe Screw	30-3648
3.5 mm x 50 mm Locking Hexalobe Screw	30-0250
3.5 mm x 55 mm Locking Hexalobe Screw	30-0251
3.5 mm x 60 mm Locking Hexalobe Screw	30-0252
3.5 mm x 65 mm Locking Hexalobe Screw	30-0253